



Specialised
Orthotic
Services
By *drive*

support and innovation

POZIFORM® Harness Range

Harnesses, support pads and straps to aid comfort and help the client maintain posture. Compatible with non-SOS wheelchairs & seating. **Suitable for all ages including children and adults.**



specialisedorthoticservices.co.uk



Made in the UK

Harnesses and supports for control, comfort and daily posture management.

POZIFORM® Harness Range

We believe that all of our clients should be safe and secure at all times. That's why we developed the Poziform range. This collection of harnesses and support pads are designed to give optimum control and comfort to assist with daily posture management.

All Poziform equipment comes with attachments, fittings and full instructions so they can be used with all seating systems and wheelchairs, including third party systems.



Available in 3 different finishes

Standard Nylon Range

Lightweight moulded nylon fabric gives a hard wearing yet comfortable texture.

Eazi Care Vinyl Range

Suitable for more demanding clients where regular cleaning is required. Our vinyl range is wipe clean with waterproof binding to prevent moisture absorption and contamination.

Dynamic Neoprene Range



Neoprene has built in elasticity that hugs the body accurately and gives a dynamic hold.



Lower Limb Control Achilles Ankle Harness

A user friendly method of maintaining foot control.


Poziform Achilles harnesses are easy to install and adjust. All of the necessary fixings are included, allowing quick attachment to most footplates.

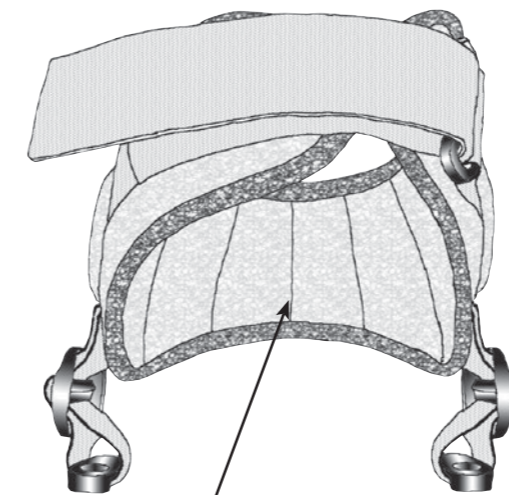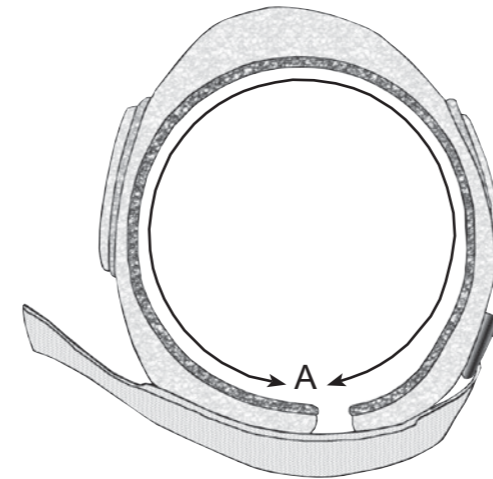

Moulded ribbing to flex around ankle & foot



Alternative Buckle Types

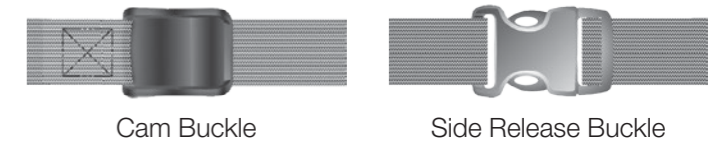

Adjustable Mounting Strap

Easily threaded through and height adjustable for secure mounting.



Supplied fixings


1. 40mm webbing slide (x4)
2. 40mm webbing bridge (x4)
3. No.8 x 12mm self tap screws (x8)
4. M4 x 20mm screws (x8)
5. M4 x Ø20 washer (x8)
6. M4 nyloc nut (x8)

| Styles | A (min) | A (max) | Product Codes | | |
|----------------------|---------|---------|-----------------|------------|---------------------|
| | | | Standard Velcro | Cam Buckle | Side Release Buckle |
| Size 1 (S) Achilles | 150 | 190 | SOS80000 | SOS80083 | SOS80087 |
| Size 2 (M) Achilles | 180 | 230 | SOS80001 | SOS80084 | SOS80088 |
| Size 3 (L) Achilles | 220 | 280 | SOS80002 | SOS80085 | SOS80089 |
| Size 4 (XL) Achilles | 280 | 340 | SOS80003 | SOS80086 | SOS80090 |

Lower Limb Control Foot Straps

A user friendly method of maintaining foot control.

Poziform Foot Straps are easy to install and adjust. They attach to most footrests, they can also be used with our range of universal comfort pads to prevent chafing.

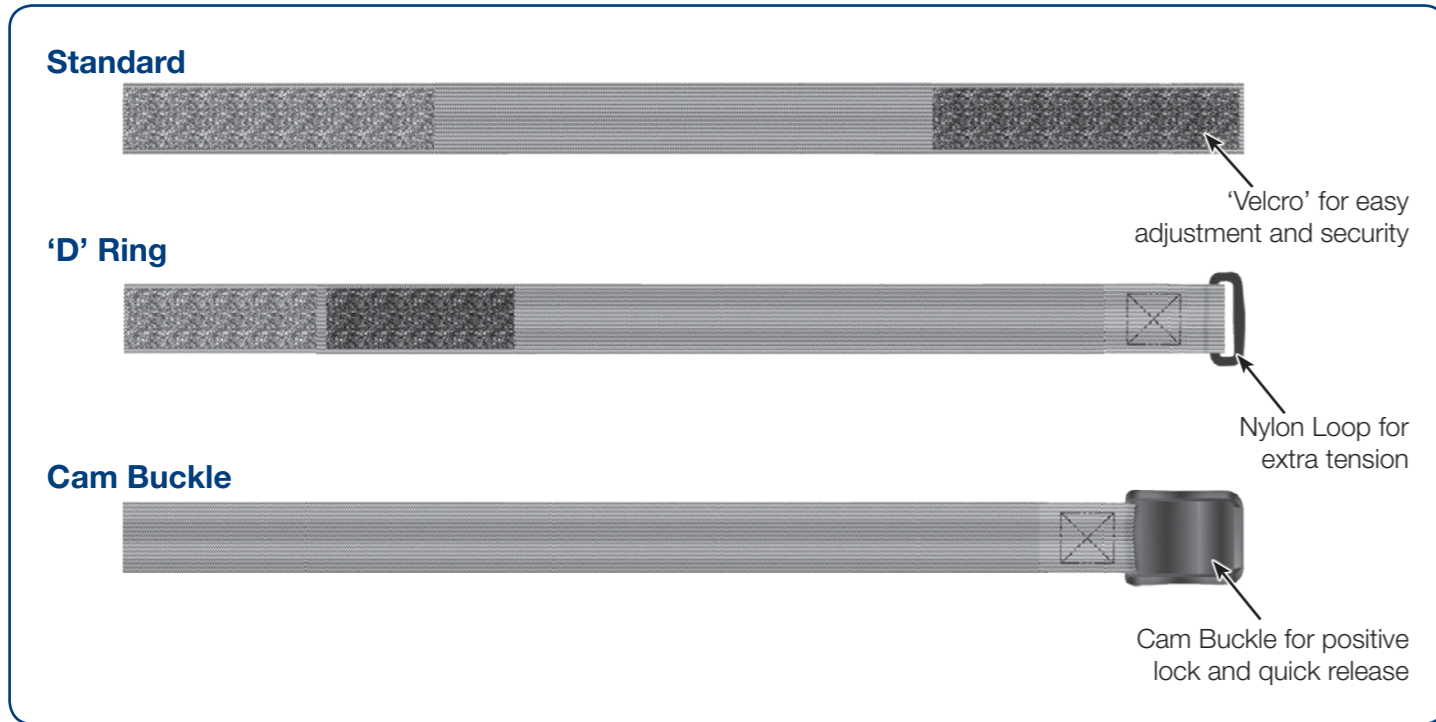

| Styles | Product Codes |
|------------|---------------|
| Standard | SOS80058 |
| 'D' Ring | SOS80059 |
| Cam Buckle | SOS80060 |

Please note; All straps are 25mm (1") wide x 585mm (23") long and supplied in pairs


Universal Comfort Pads

Straps can be used singularly, in pairs or in conjunction with our comfort pads as shown

Lower Limb Control Calf Panel

Wheelchair frameworks can cause a hazard for the legs and feet of an active client. Poziform Calf Panels help prevent the client pushing their legs underneath their seat whilst providing a comfortable support.

Quick to install and transfer from chair to chair, Poziform Calf Panels do not prevent the wheelchair from folding. Available in a range of materials.

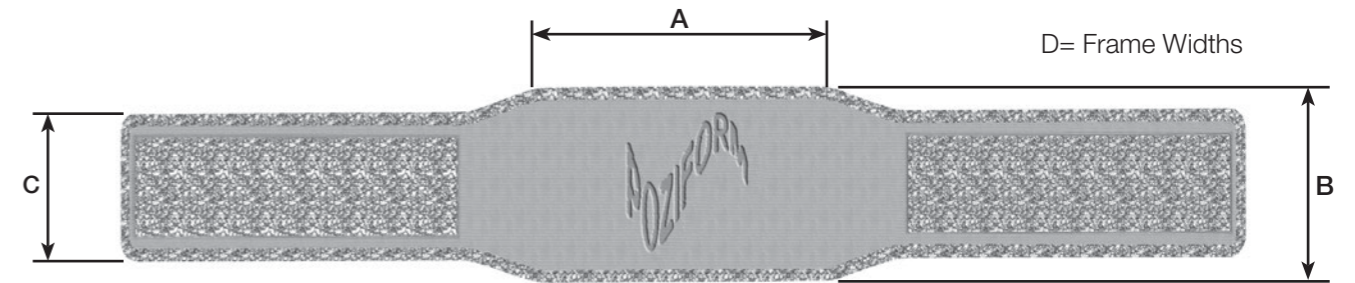


| Styles | Dimensions mm | | | | | Product Codes | | |
|-----------------------|--|-----|-----|--------------|---------|----------------|------------------|---------------|
| | A | B | C | Frame Widths | | Standard Nylon | Dynamic Neoprene | Spacer Fabric |
| | | | | D (min) | D (max) | | | |
| Size 2 (S) Calf Panel | 330 | 115 | 90 | 325 | 430 | SOS80035 | SOS80102 | SOS80100 |
| Size 3 (M) Calf Panel | 380 | 150 | 115 | 360 | 490 | SOS80036 | SOS80103 | SOS80101 |
| Size 4 (L) Calf Panel | 475 | 165 | 125 | 460 | 625 | N/A | SOS10459 | N/A |
| Custom | Custom Sizes Available - Please contact sales department | | | | | | | |

Pelvic Control Pelvic Harness

A simple method to secure the pelvis when seated. This single part pelvic harness tightens at the centre buckle.

Effective control of the pelvis in sitting posture is a key component in maintaining postural stability and control. SOS Pelvic harnesses are designed for use with special seating and standard wheelchair seating.

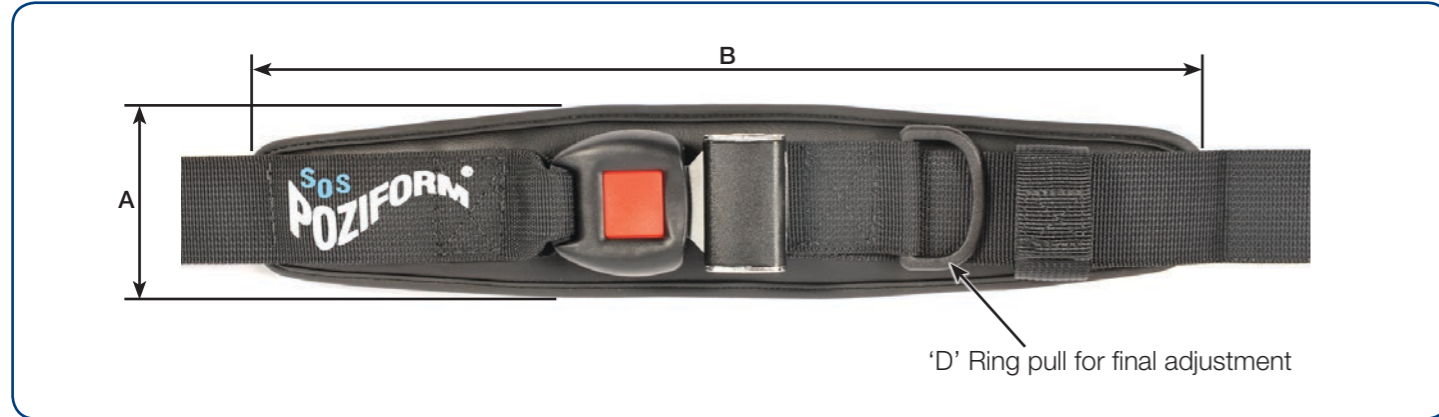

Specification

| | Dimensions mm | | Product Codes | | | | |
|----------------------------|---------------|-----|----------------------|---------------------------|-----------------|---------------------|---------------------------|
| | A | B | Standard Push Buckle | Quick Release Push Button | Aircraft Buckle | Side Release Buckle | Twin Lock Security Buckle |
| Standard Nylon | | | | | | | |
| Size 1 (XS) Pelvic Harness | 50 | 230 | N/A | N/A | N/A | SOS80015 | SOS80122 |
| Size 2 (S) Pelvic Harness | 70 | 310 | SOS80016 | SOS80113 | SOS80116 | SOS80119 | SOS80123 |
| Size 3 (M) Pelvic Harness | 70 | 365 | SOS80017 | SOS80114 | SOS80117 | SOS80120 | SOS80124 |
| Size 4 (L) Pelvic Harness | 70 | 415 | SOS80018 | SOS80115 | SOS80118 | SOS80121 | SOS80125 |


| | Dimensions mm | | Product Codes | | | | |
|----------------------------|---------------|-----|----------------------|---------------------------|-----------------|---------------------|---------------------------|
| | A | B | Standard Push Buckle | Quick Release Push Button | Aircraft Buckle | Side Release Buckle | Twin Lock Security Buckle |
| Eazi Care Vinyl | | | | | | | |
| Size 1 (XS) Pelvic Harness | 50 | 230 | N/A | N/A | N/A | SOS10399 | SOS10483 |
| Size 2 (S) Pelvic Harness | 70 | 310 | SOS10400 | SOS10470 | SOS10473 | SOS10480 | SOS10484 |
| Size 3 (M) Pelvic Harness | 70 | 365 | SOS10401 | SOS10471 | SOS10474 | SOS10481 | SOS10485 |
| Size 4 (L) Pelvic Harness | 70 | 415 | SOS10402 | SOS10472 | SOS10475 | SOS10482 | SOS10486 |

POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Pelvic Control 2 Part Single Pull Pelvic Harness

The 2 part pelvic harness is split down the middle at the buckle for more flexibility and adjustment. Tightens from one side.

Effective control of the pelvis in sitting posture is a key component in maintaining postural stability and control. SOS Pelvic harnesses are designed for use with special seating and standard wheelchair seating.

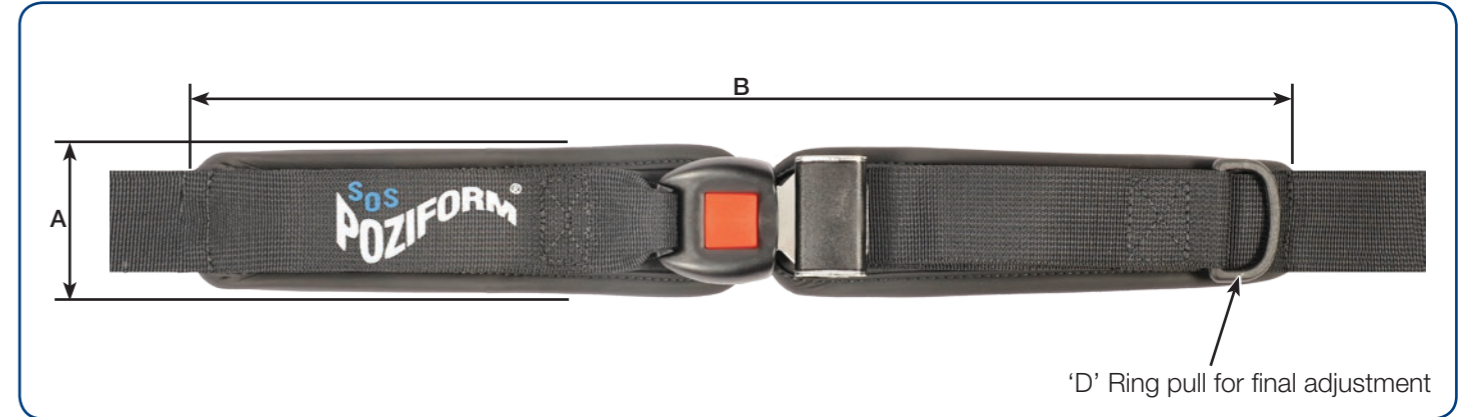

Specification

| | Dimensions mm | | | Product Codes | | | | |
|-----------------------------------|---------------|---------|---------|----------------------|---------------------------|-----------------|---------------------|---------------------------|
| | A | B (min) | B (max) | Standard Push Buckle | Quick Release Push Button | Aircraft Buckle | Side Release Buckle | Twin Lock Security Buckle |
| Standard Nylon | | | | | | | | |
| Size 1 (XS) 2-Part Pelvic Harness | 55 | 310 | 385 | N/A | N/A | N/A | SOS80027 | SOS80144 |
| Size 2 (S) 2-Part Pelvic Harness | 65 | 360 | 445 | N/A | N/A | N/A | SOS80061 | SOS80145 |
| Size 3 (M) 2-Part Pelvic Harness | 65 | 460 | 560 | SOS80028 | SOS80135 | SOS80138 | SOS80141 | SOS80146 |
| Size 4 (L) 2-Part Pelvic Harness | 75 | 560 | 685 | SOS80029 | SOS80136 | SOS80139 | SOS80142 | SOS80147 |
| Size 5 (XL) 2-Part Pelvic Harness | 75 | 620 | 770 | SOS80030 | SOS80137 | SOS80140 | SOS80143 | SOS80148 |


| | Dimensions mm | | | Product Codes | | | | |
|-----------------------------------|---------------|---------|---------|----------------------|---------------------------|-----------------|---------------------|---------------------------|
| | A | B (min) | B (max) | Standard Push Buckle | Quick Release Push Button | Aircraft Buckle | Side Release Buckle | Twin Lock Security Buckle |
| Eazi Care Vinyl | | | | | | | | |
| Size 1 (XS) 2-Part Pelvic Harness | 55 | 260 | 335 | N/A | N/A | N/A | SOS10403 | SOS10618 |
| Size 2 (S) 2-Part Pelvic Harness | 60 | 315 | 400 | N/A | N/A | N/A | SOS10429 | SOS10619 |
| Size 3 (M) 2-Part Pelvic Harness | 65 | 410 | 510 | SOS10404 | SOS10600 | SOS10603 | SOS10606 | SOS10620 |
| Size 4 (L) 2-Part Pelvic Harness | 65 | 515 | 640 | SOS10405 | SOS10601 | SOS10604 | SOS10607 | SOS10621 |
| Size 5 (XL) 2-Part Pelvic Harness | 75 | 620 | 770 | SOS10406 | SOS10602 | SOS10605 | SOS10608 | SOS10622 |

POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Pelvic Control 2 Part Dual Pull Pelvic Harness

For clients who need a more positive hold on the pelvis. 2 part pelvic harness is split down the middle at the buckle and tightens from both sides.

Effective control of the pelvis in sitting posture is a key component in maintaining postural stability and control. SOS Pelvic harnesses are designed for use with special seating and standard wheelchair seating.

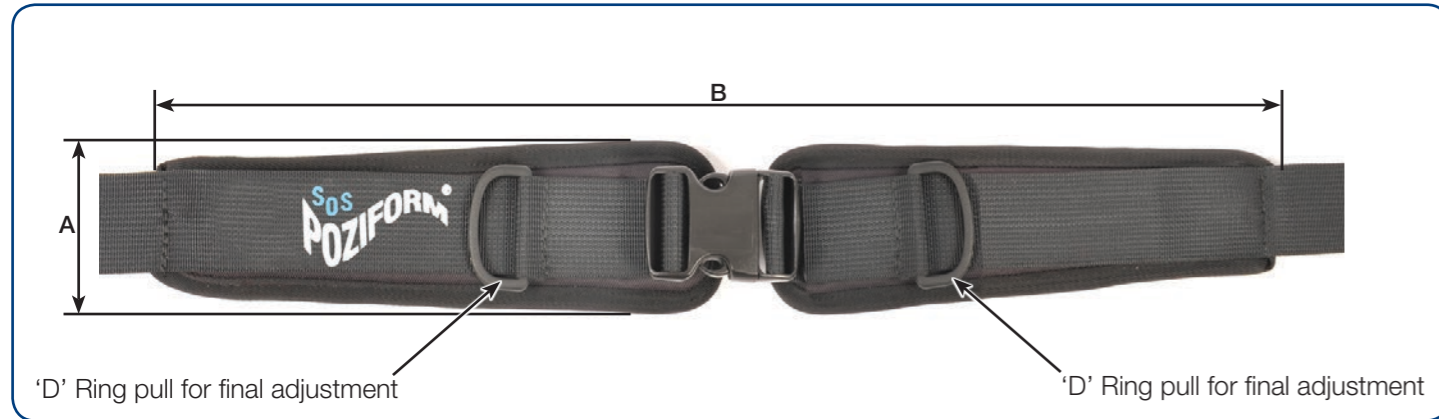

Specification

| Standard Nylon | Dimensions mm | | | Product Codes |
|-----------------------------------|---------------|---------|---------|---------------------|
| | A | B (min) | B (max) | Side Release Buckle |
| Size 1 (XS) 2-Part Pelvic Harness | 55 | 310 | 410 | SOS80031 |
| Size 2 (S) 2-Part Pelvic Harness | 60 | 360 | 475 | SOS80062 |
| Size 3 (M) 2-Part Pelvic Harness | 65 | 460 | 585 | SOS80032 |
| Size 4 (L) 2-Part Pelvic Harness | 65 | 560 | 710 | SOS80033 |
| Size 5 (XL) 2-Part Pelvic Harness | 75 | 620 | 770 | SOS80034 |


| Eazi Care Vinyl | Dimensions mm | | | Product Codes |
|-----------------------------------|---------------|---------|---------|---------------------|
| | A | B (min) | B (max) | Side Release Buckle |
| Size 1 (XS) 2-Part Pelvic Harness | 55 | 260 | 360 | SOS10407 |
| Size 2 (S) 2-Part Pelvic Harness | 60 | 315 | 425 | SOS10430 |
| Size 3 (M) 2-Part Pelvic Harness | 65 | 410 | 535 | SOS10408 |
| Size 4 (L) 2-Part Pelvic Harness | 65 | 515 | 665 | SOS10409 |
| Size 5 (XL) 2-Part Pelvic Harness | 75 | 620 | 770 | SOS10410 |

POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Pelvic Control 2 Part Dual Pull 4 Point Pelvic Harness

2 part dual pull 4 point pelvic harness is split down the middle at the buckle and tightens from both sides. Featuring 2 additional attachment points for extra security. Suitable for clients with substantial movement and moderate to severe positioning needs.

Effective control of the pelvis in sitting posture is a key component in maintaining postural stability and control. SOS Pelvic harnesses are designed for use with special seating and standard wheelchair seating.

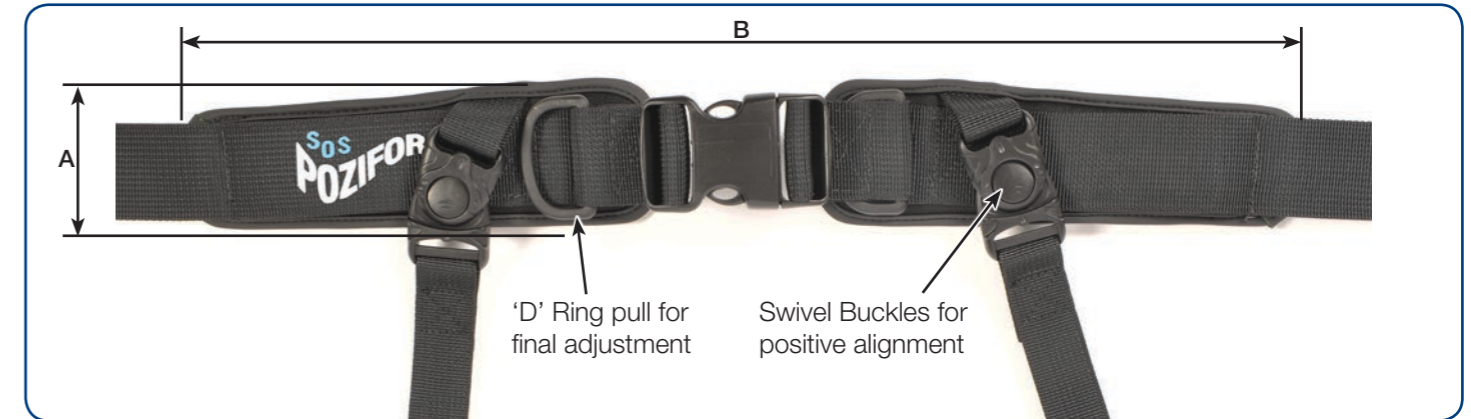

Specification

| Standard Nylon | Dimensions mm | | | Product Codes |
|-----------------------------------|---------------|---------|---------|---------------------|
| | A | B (min) | B (max) | Side Release Buckle |
| Size 1 (XS) 2-Part Pelvic Harness | 55 | 310 | 410 | SOS80064 |
| Size 2 (S) 2-Part Pelvic Harness | 60 | 360 | 475 | SOS80065 |
| Size 3 (M) 2-Part Pelvic Harness | 65 | 460 | 585 | SOS80066 |
| Size 4 (L) 2-Part Pelvic Harness | 65 | 560 | 710 | SOS80067 |
| Size 5 (XL) 2-Part Pelvic Harness | 75 | 620 | 770 | SOS80068 |


| Eazi Care Vinyl | Dimensions mm | | | Product Codes |
|-----------------------------------|---------------|---------|---------|---------------------|
| | A | B (min) | B (max) | Side Release Buckle |
| Size 1 (XS) 2-Part Pelvic Harness | 55 | 260 | 360 | SOS10431 |
| Size 2 (S) 2-Part Pelvic Harness | 60 | 315 | 425 | SOS10432 |
| Size 3 (M) 2-Part Pelvic Harness | 65 | 410 | 535 | SOS10433 |
| Size 4 (L) 2-Part Pelvic Harness | 65 | 515 | 665 | SOS10434 |
| Size 5 (XL) 2-Part Pelvic Harness | 75 | 620 | 770 | SOS10435 |

POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Pelvic Control Groin Harness

The groin harness helps fix the position of the pelvis without putting any pressure on the abdominal area. Ideal for when clients are prone to sliding forwards in their wheelchair seat.

Available in wipe clean Eazi Care Vinyl only.

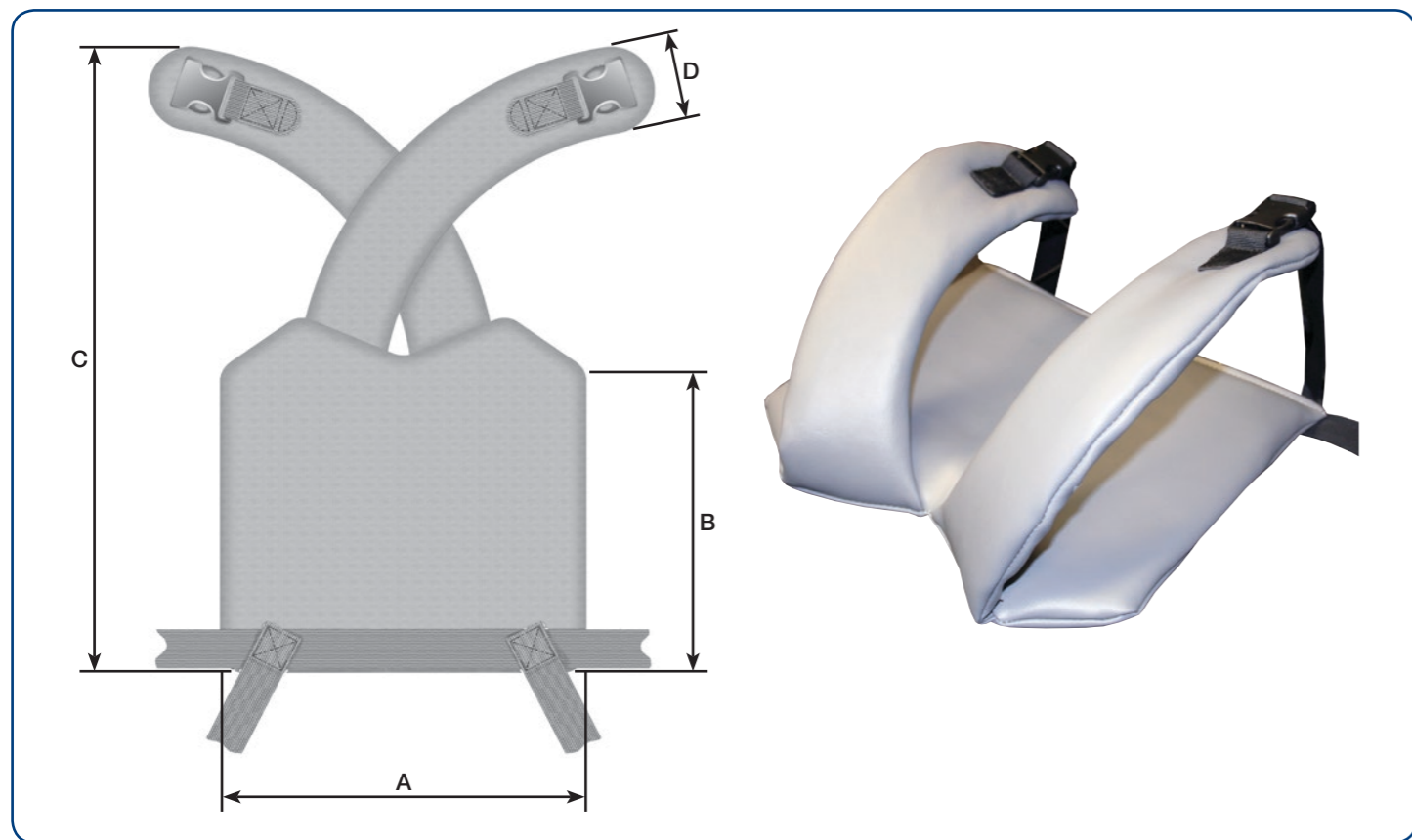

Specification

| Styles | Dimensions mm | | | | Product Codes |
|---------------------------|---------------|-----|-----|----|-----------------|
| | A | B | C | D | Eazi Care Vinyl |
| Size 1 (S) Groin Harness | 210 | 150 | 350 | 50 | SOS10490 |
| Size 2 (M) Groin Harness | 250 | 220 | 450 | 50 | SOS10491 |
| Size 3 (L) Groin Harness | 300 | 250 | 570 | 70 | SOS10492 |
| Size 4 (XL) Groin Harness | 360 | 290 | 720 | 70 | SOS10493 |

Buckle Options



Groin Strap Buckles




Groin Harness Fixing Buckle

POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Upper Body Control Cross Harness

Upper body harnesses help maintain shoulder and chest alignment, stopping the client from leaning forward in their seat. The 'X' harness, sometimes known as 'butterfly harness' is a proven design for helping maintain postural control and stability.

Featuring bottom swivel anchor points for improved comfort. Supplied with front release buckles.

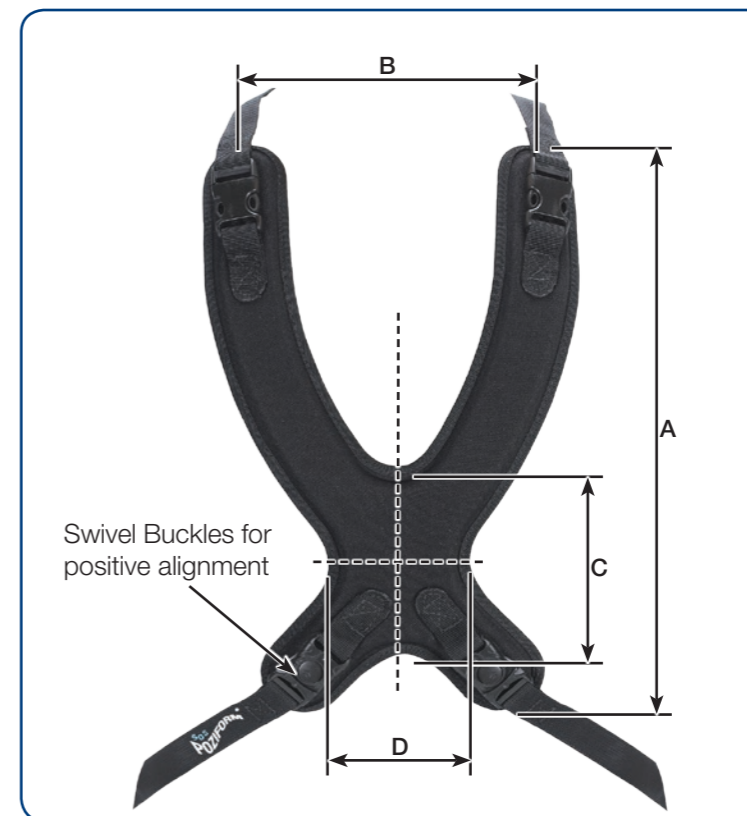

Specification

| Styles | Dimensions mm | | | | Product Codes | | |
|---------------------------|---------------|-----|-----|-----|----------------|-----------------|------------------|
| | A | B | C | D | Standard Nylon | Eazi Care Vinyl | Dynamic Neoprene |
| Size 0 (XS) Cross Harness | 205 | 135 | 100 | 65 | SOS80008 | SOS10412 | SOS80011 |
| Size 1 (S) Cross Harness | 300 | 145 | 105 | 75 | SOS80004 | SOS10494 | SOS80069 |
| Size 2 (M) Cross Harness | 385 | 170 | 115 | 85 | SOS80005 | SOS10495 | SOS80070 |
| Size 3 (L) Cross Harness | 455 | 215 | 150 | 100 | SOS80006 | SOS10496 | SOS80071 |
| Size 4 (XL) Cross Harness | 525 | 260 | 160 | 105 | SOS80007 | SOS10497 | SOS80072 |
| LADIES | | | | | | | |
| Size 2 (M) Cross Harness | 375 | 190 | 155 | 40 | SOS80012 | SOS10498 | SOS80073 |
| Size 3 (L) Cross Harness | 440 | 220 | 160 | 60 | SOS80013 | SOS10499 | SOS80074 |

Buckle Options



All Upper Strap Buckles




All Cross Harness Lower Strap Buckles

POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Upper Body Control Contoured Shoulder Harness

Upper body harnesses help maintain shoulder and chest alignment, stopping the client from leaning forward in their seat. The Poziform Contoured Shoulder Harness is designed for easy adjustment and quick release.

Featuring bottom swivel anchor points for improved comfort.

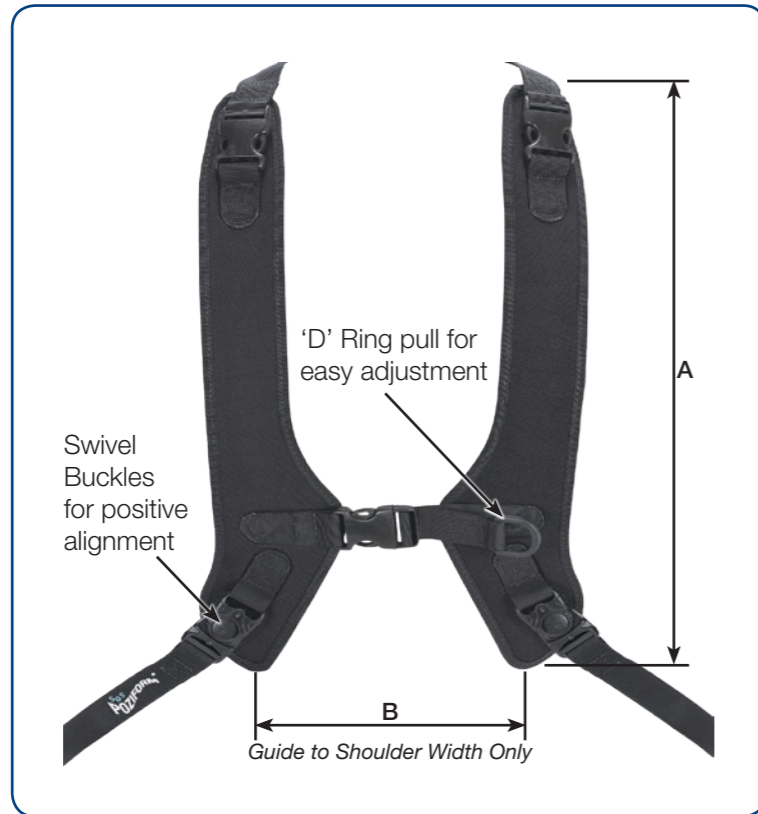

Specification

| Styles | Dimensions mm | | | Product Codes | | |
|--|---------------|---------|---------|----------------|-----------------|------------------|
| | A | B (min) | B (max) | Standard Nylon | Eazi Care Vinyl | Dynamic Neoprene |
| Size 1 (XS) Contoured Shoulder Harness | 205 | 100 | 170 | SOS80047 | SOS10416 | SOS80075 |
| Size 2 (S) Contoured Shoulder Harness | 255 | 120 | 180 | SOS80048 | SOS10417 | SOS80076 |
| Size 3 (M) Contoured Shoulder Harness | 360 | 120 | 260 | SOS80049 | SOS10418 | SOS80077 |
| Size 4 (L) Contoured Shoulder Harness | 405 | 140 | 280 | SOS80063 | SOS10428 | SOS80078 |
| Size 5 (XL) Contoured Shoulder Harness | 460 | 155 | 290 | SOS80050 | SOS10419 | SOS80079 |

Buckle Options



Upper Strap Buckles



Upper Strap Buckles




All Lower Strap Buckles

POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Upper Body Control Bib Harness

Upper body harnesses help maintain shoulder and chest alignment, stopping the client from leaning forward in their seat. The Bib harness offers a one-piece solution for upper body positioning. Available in wipe clean Eazi Care Vinyl and Dynamic Neoprene.

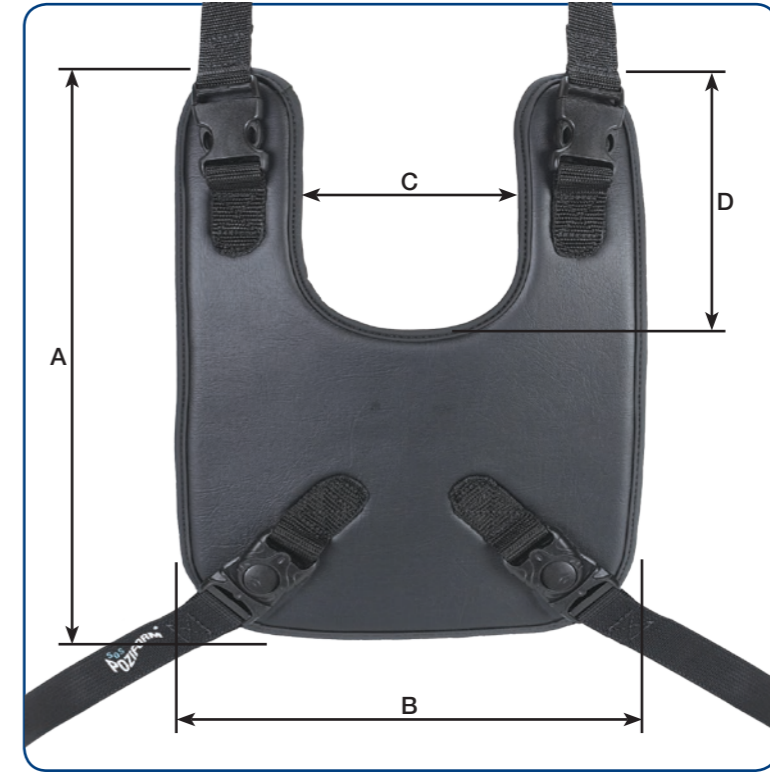

Specification

| Styles | Dimensions mm | | | | Product Codes | |
|----------------|---------------|-----|-----|-----|-----------------|------------------|
| | A | B | C | D | Eazi Care Vinyl | Dynamic Neoprene |
| Size 1 (S) Bib | 230 | 190 | 110 | 100 | SOS10505 | SOS80080 |
| Size 2 (M) Bib | 305 | 210 | 110 | 140 | SOS10506 | SOS80081 |
| Size 3 (L) Bib | 345 | 260 | 160 | 155 | SOS10507 | SOS80082 |

Buckle Options



All Upper Strap Buckles



All Lower Strap Buckles

POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Upper Body Control 5 Point Harness

Upper body harnesses help maintain shoulder and chest alignment, stopping the client from leaning forward in their seat. Available with 25mm or 40mm straps, the 5 Point Harness is supportive and durable. Both versions are fitted with split shoulder straps for easy access around the client.

Supplied in black webbing only with central button release. Optional buckle/sternum pad available if required.




Specification

| Styles | Product Codes |
|--------|---------------|
| 25mm | SOS30028 |
| 40mm | SOS30029 |

POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Upper Body Control Harness Fixing Kit

The SOS harness fixing kit is designed to add additional adjustment when fitting harnesses to canvas wheelchairs. Effective at maintaining the position of cross harnesses where standard fitting is not possible. SOS also have the ability to manufacture harness kits with different buckle styles to suit other types of upper body harnesses.

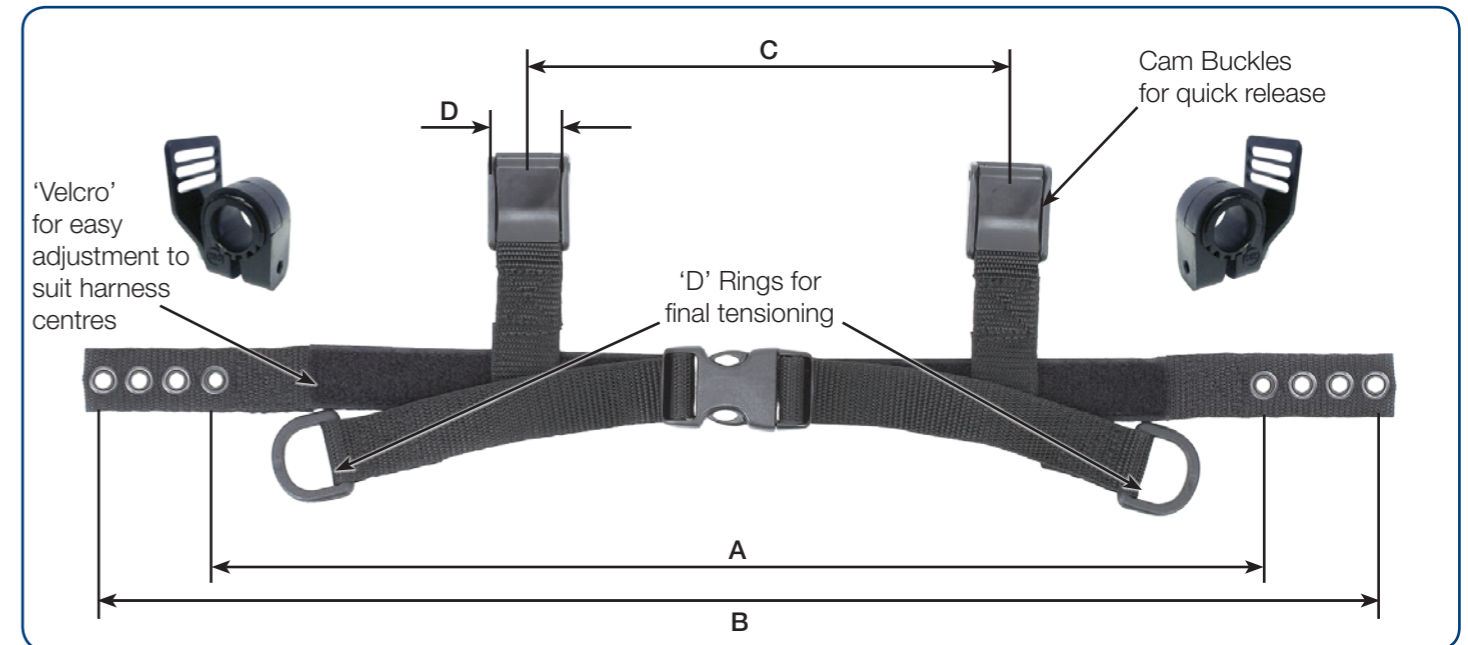

Specification


| Styles | Dimensions mm | | | | | Product Codes |
|-----------------------------------|---------------|-----|---------|---------|----|---------------|
| | A | B | C (min) | C (max) | D | |
| Size 1 (S) | | | | | | |
| Harness Fixing Kit (At Min Width) | 275 | 375 | 85 | 200 | 25 | SOS80056 |
| Harness Fixing Kit (At Max Width) | 335 | 435 | 120 | 255 | 25 | |
| Size 2 (M) | | | | | | |
| Harness Fixing Kit (At Min Width) | 375 | 475 | 95 | 290 | 25 | SOS80057 |
| Harness Fixing Kit (At Max Width) | 535 | 635 | 220 | 450 | 25 | |

POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Upper Body Control Dynamic Chest Strap

The SOS Chest Strap is manufactured from neoprene for flexible trunk support and positioning.

Featuring a usable length of 150cm (Xlong 165cm) and simple Velcro fastening for easy adjustment.

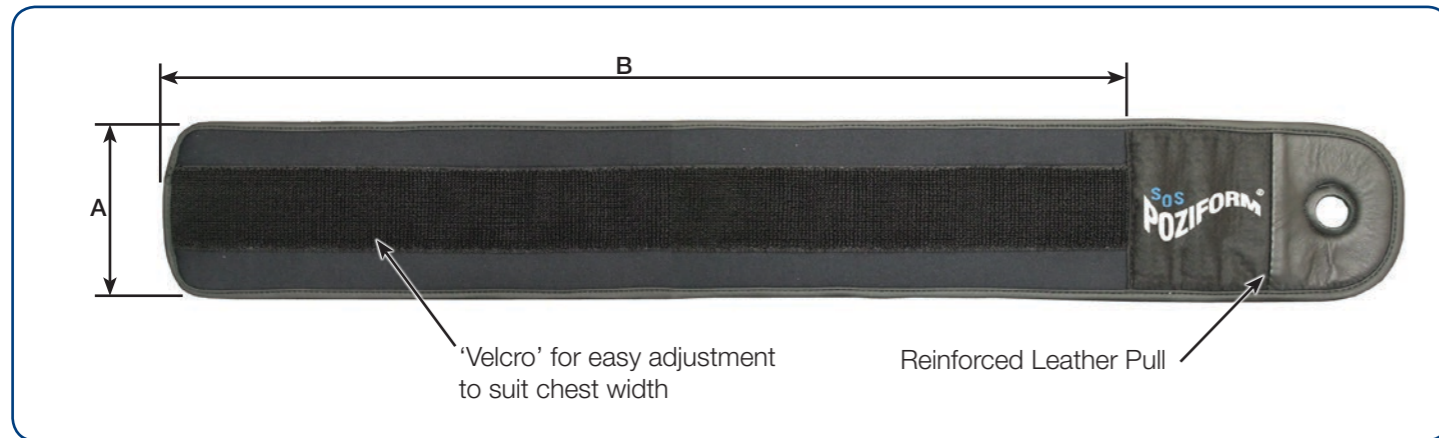

Specification

| Styles | Dimensions mm | | Product Codes |
|------------------------------|---------------|------|------------------|
| | A | B | Dynamic Neoprene |
| Small - Dynamic Chest Strap | 75 | 1500 | SOS10454 |
| Medium - Dynamic Chest Strap | 100 | 1500 | SOS10455 |
| Large - Dynamic Chest Strap | 125 | 1500 | SOS10456 |
| XLong - Dynamic Chest Strap | 125 | 1650 | SOS10457 |

POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE


Upper Body Control Poziflex Chest Harness

The Poziflex Chest Strap, manufactured from neoprene is a comfortable elastic pad with just the right amount of stretch.

Featuring a side release buckle on both sides for accurate adjustment. Comes complete with fixings.

Sizing

Measure the client around the front and side at the height the strap will be worn and select a size that will cover the front and wrap half way around the side of the trunk. This will allow space for adjustment of the buckles.

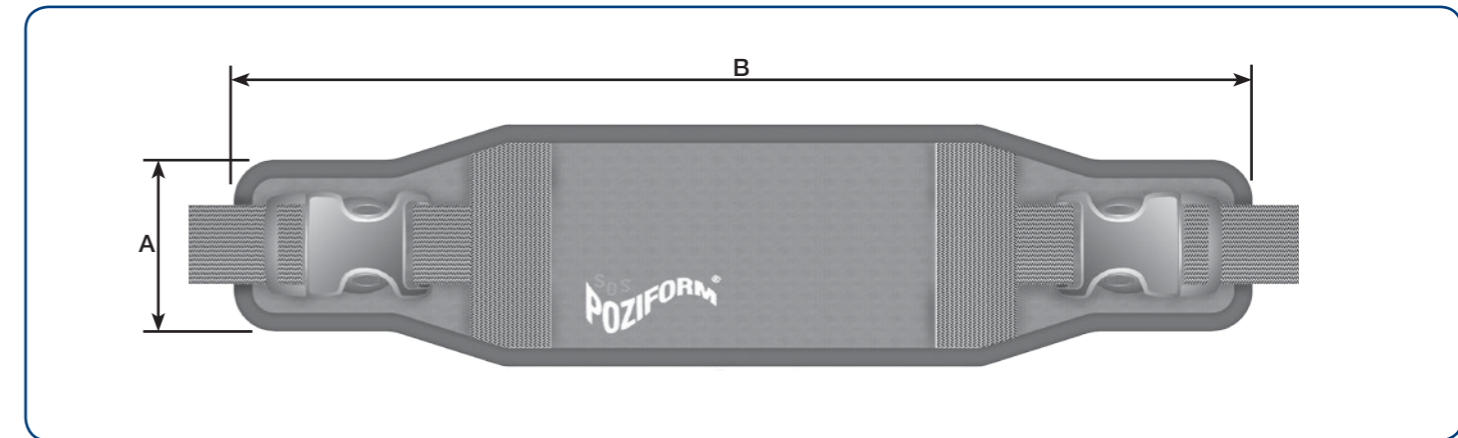

Specification

| Styles | Dimensions mm | | Product Codes |
|---------------------------------|---------------|-----|------------------|
| | A | B | Dynamic Neoprene |
| XSmall - Poziflex Chest Harness | 75 | 320 | SOS10508 |
| Small - Poziflex Chest Harness | 95 | 400 | SOS10509 |
| Medium - Poziflex Chest Harness | 110 | 475 | SOS10510 |
| Large - Poziflex Chest Harness | 120 | 550 | SOS10511 |
| XLarge - Poziflex Chest Harness | 135 | 620 | SOS10512 |


POZIFORM HARNESSSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Comfort Pads


Universal Pads

SOS Universal Comfort Pads can be used with any standard straps or webbing to prevent chafing and improve comfort. They can also help improve the location of client's belts and straps preventing them from moving.


Standard

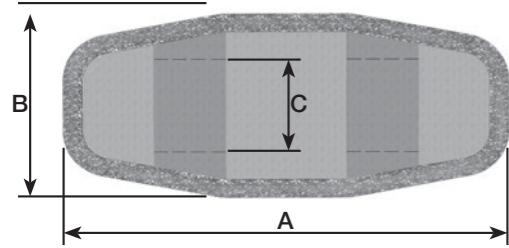

Tapered

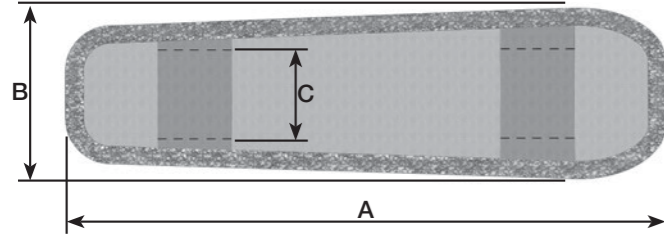

Maxi

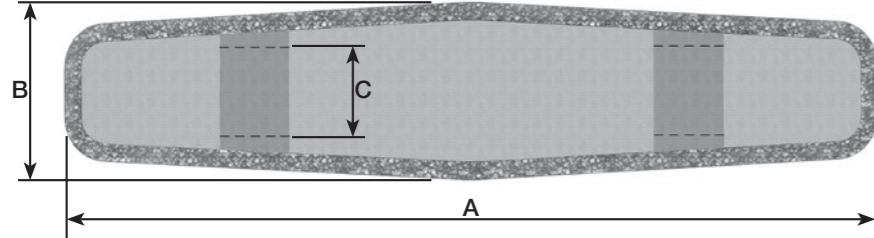

C= Maximum width of webbing



Specification

| Styles | Dimensions mm | | | Dimensions mm | | | Product Codes | | |
|---------------------------|-----------------------------------|----|----|-----------------|----|----|----------------|-----------------|------------------|
| | Standard Nylon & Dynamic Neoprene | | | Eazi Care Vinyl | | | Standard Nylon | Eazi Care Vinyl | Dynamic Neoprene |
| | A | B | C | A | B | C | | | |
| STANDARD | | | | | | | | | |
| Size 1 (S) Universal Pad | 135 | 55 | 25 | 135 | 55 | 25 | SOS80019 | SOS10420 | SOS10460 |
| Size 2 (M) Universal Pad | 165 | 75 | 40 | 165 | 75 | 40 | SOS80020 | SOS10421 | SOS10461 |
| TAPERED | | | | | | | | | |
| Size 1 (XS) Universal Pad | 155 | 55 | 25 | 135 | 55 | 25 | SOS80021 | SOS10422 | SOS10465 |
| Size 2 (S) Universal Pad | 180 | 65 | 40 | 205 | 65 | 40 | SOS80022 | SOS10423 | SOS10466 |
| Size 3 (M) Universal Pad | 230 | 65 | 40 | 255 | 65 | 40 | SOS80023 | SOS10424 | SOS10467 |
| Size 4 (L) Universal Pad | 280 | 75 | 40 | 305 | 75 | 40 | SOS80024 | SOS10425 | SOS10468 |
| MAXI | | | | | | | | | |
| Size 2 (S) Universal Pad | 310 | 70 | 40 | 310 | 70 | 40 | SOS80025 | SOS10426 | SOS10487 |
| Size 3 (M) Universal Pad | 365 | 70 | 40 | 365 | 70 | 40 | SOS80026 | SOS10427 | SOS10488 |

POZIFORM HARNESSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Fittings and Fixings

Frame Mounting clamps

SOS Tube Clamps are recommended to attach all Poziform harnesses securely to standard wheelchair mounts.

Manufactured to fit 25mm, 22mm and 19mm diameter frame tubes via adapters provided.

Specification

| Styles | Product Codes |
|----------------------------|---------------|
| Frame Clamp (25mm Webbing) | SOS10451 |
| Frame Clamp (40mm Webbing) | SOS10452 |



Bar Slides

| Size | Product Codes |
|-------------------|---------------|
| 25mm (Pack of 10) | SOS80597 |
| 40mm (Pack of 10) | SOS80598 |



Swivel Webbing Guides

| Size | Product Codes |
|-------------------|---------------|
| 25mm (Pack of 10) | SOS80582 |
| 40mm (Pack of 10) | SOS80583 |



Buckle Options



Standard Push Buckle



Aircraft Buckle



Twin Lock Security Buckle



Quick Release Push-Button Buckle



Side Release Buckle


POZIFORM HARNESSES ARE NOT SUITABLE AS SAFETY STRAPS ON A MOVING VEHICLE

Useful Measurements For Harness Selection




Ankle
Circumference

Ankle Harness

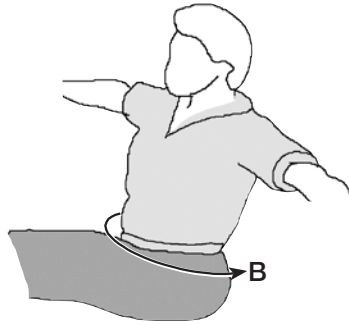

Measured from one side of
the pelvis to the other side

Pelvic Harness

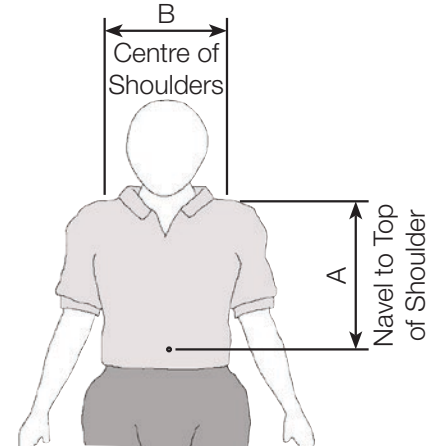

Upper Body Harness

Safety Information

Poziform harnesses are designed to provide postural support only and are not to be used as a passenger restraint for transportation. A separate passenger restraint should always be used in line with the recommendation of the wheelchair manufacturer.

Upper body harnesses should be fitted no higher than the sternum (mid chest). Allow enough space between neck and the harness to ensure it does not pose a choking risk.

Where an upper body harness is being used, always use in conjunction with an appropriately fitted pelvic belt to ensure the pelvis is well supported and minimize the risk of the client sliding out of the seat.

Ensure the harnesses and pelvic belts hold the client securely and are comfortable.

Achilles ankle harnesses and foot straps must be fitted and worn over everyday footwear.

Need more info or advice?

For more information, advice or to book a demonstration call: 0044 (0)1283 812860

Email us: enquiries@specialorthotic.com

Visit our website: www.specialisedorthoticservices.co.uk



Specialised Orthotic Services
Unit 127/8
Fauld Industrial Park
Tutbury
Staffordshire
DE13 9HS

Tel: 0044 (0)1283 812860
Fax: 0044 (0)1283 520401
Email: enquiries@specialorthotic.com
Web: www.specialisedorthoticservices.co.uk

