



Removable mattress cover and cell system



Quick connector

CPR


To deflate the mattress in the event of an emergency, rotate the quick connector ANTI-CLOCKWISE to the CPR position

Automatic panel lock out

Locks the panel to prevent unauthorised setting changes. To unlock press and hold for 3 seconds.

Max firm mode

This mode ensures the mattress reaches its maximum pressure. After 20 minutes the pump will automatically revert back to the preset comfort setting.

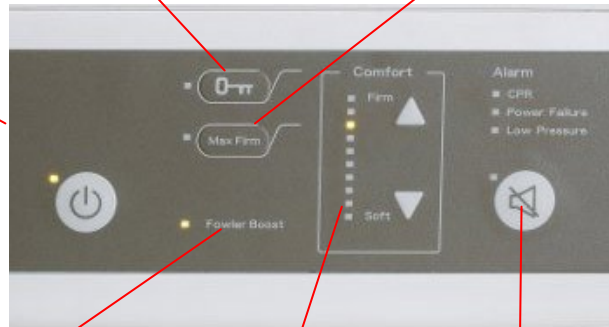

Automatic Fowler Boost

This will automatically adjust the pressure to accommodate the semi-recumbent (sitting) position

Comfort Control

10 adjustable comfort levels from Soft to Firm

Alarm mute

This will temporarily silence the CPR and low pressure alarms. The alarms will resume if the problem is not resolved within 3 minutes


QUICK SET UP AND OPERATION GUIDELINES

Power failure alarm

If the pump detects a power failure the power failure LED will illuminate and the alarm will sound

CPR alarm

During CPR procedure the CPR LED will illuminate and the alarm will sound

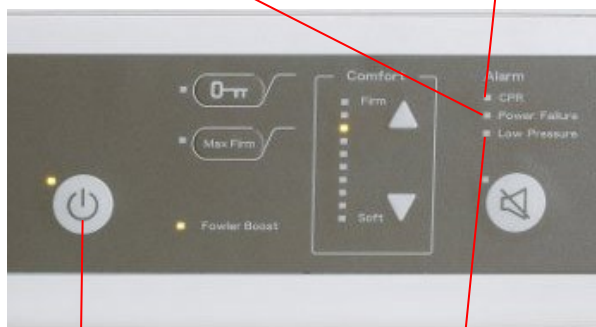

Standby button

Low pressure indicator

If the pump detects a low pressure the low pressure LED will illuminate and the alarm will sound.

Set up and Operation

1. Remove the existing mattress and place the Ambience system directly onto the bed frame with the cells uppermost and the inflation tube at the foot end of the bed.
2. Ensure the Fowler boost sensor pad is secured in the designated position (inside the bottom section of the mattress)
3. Connect the Quick connector to the pump unit and rotate clockwise to the Lock/On position
4. Connect the Fowler boost adaptor to the side of the pump unit
5. Connect pump to the mains power and turn on via the green switch on the side of the pump
6. The pressure in the mattress can be adjusted by selecting one of the 10 comfort levels on the control panel.

Alarm Mute

The alarm mute button will silence the low pressure and CPR alarms for 3 minutes, if the problem is not resolved the alarm will resume. During a power failure situation, pressing this button will turn off power failure alarm and the LED completely.

Please refer to Troubleshooting for more information or contact Park House Healthcare.