

Instructions:

1. Fit for the normal user.



1.1 Put the safety harness on the backrest



1.2 Pull the safety harness through the cross braces.



1.3 Fasten the belt in the back.



1.4 When the user sits properly, adjust the seat belt.



1.5 Adjust the seat belt in the back.

2. Fit for the patient who has stroke



1.1 Put the safety harness on the backrest



1.2 Pull the safety harness through the cross braces.



1.3 Cross the belt and fasten it.



1.4 When the user sits properly, adjust the seat belt.



1.5 Adjust the seat belt in the back.