

# HOW TO FIT A CLICK & GO LITE.



## STEP 1.

Once you have unpacked your Click & Go Lite, locate the locking mechanism as shown in the picture.

Untighten the two Allen bolts.

Once removed, proceed to Step 2.



**STEP 2.**

Place the locking mechanism on the lower side frame, just in front of the rear axle plate, as shown in the picture.

Tighten the two bolts to hold the locking mechanism to the side frame.

Ensure the locking mechanism grips on to the cylindrical tube. This will need to be very tight.

Proceed to STEP 3.





**STEP 3.**

Please ensure the two locking mechanism are in line if they are out of line your Click & Go Lite will not drive in a straight line.

Please see the two fitted locking mechanisms in place.

Proceed to Step 4.



Step 4.

Locate the silver tube included with the Click & Go Lite.

Please note that this can be extended by untightening the black star shaped knob.

Proceed to Step 5.





Step 5.

On the unit itself, there is a Black plate that fixes the extendable silver bar to the unit.

Please loosen the 4 Allen head bolts.

When removed, please proceed to Step 6.



Step 6.

Place the silver tube found in step 4, through the Black Plate.

Proceed to tighten the 4 Allen head bolts found in step 5 on to the Black Plate.

It is recommended that the unit is centralised on the silver fixation tube before fitting to the chair. Please use the black star knob to tighten the silver fixation tube in place.

Once completed, proceed to Step 7.



**Step 7.**

To attach the unit to the locking mechanisms, please pull down on the yellow handles.

This releases the spring loaded locking mechanism enabling you to the silver tube.

Please use the picture shown for guidance.

Once complete, proceed to step 8.





Step 8.

Fit the battery to the unit, by connecting the black and red connectors together.

Once complete, proceed to Step 9.





#### Step 9.

In the small brown box, included in with the Click & Go Lite. There is the Hand Controller, which allows the attendant to control the speed and direction of the chair.

On the hand controller is two Allen head bolts. Please untighten the bolts.

When removed, please attach the hand controller to either the Left or Right hand side push handle.

Proceed to tighten the two Allen head bolts. This will keep the hand controller nice and sturdy.

Once tightened proceed to step 10



Step 10.

On the Click & Go unit, please find the connector as shown in the picture.

Proceed to Step 11.





**Step 11.**

Place the connector to the fitting on the bottom of the hand controller.

There is only one way this will fit.

Once fitted, tighten the silver collar to keep the connector in place.

Proceed to Step 12.



Controls direction.

Controls the speed.

Step 12.

Insert the key to the hand controller.

The green light will illuminate. If not please ensure the batteries are connected.





  
**EXCEL**  
VANOS  
Click & Go Lite