



Aerocarbon Powerchair

Attendant Controller Bracket



Part Code: 12-EC8003Z-04AB

Part: Attendant Controller Bracket & Fixings



Tools included:

- 3mm hex key (2#)

Installation Instructions:

1. Disconnect and remove the battery.
2. Remove the controller and bracket from the armrest, by loosening the knob (circled).



3. Remove the 2 screws attaching the controller to the bracket, using a 3mm hex key



4. Attach the new bracket to the controller, using a 3mm hex key.



5. Attach the attendant controller bracket onto the push handle. Secure and clamp to the handle using two 3mm hex keys.



6. Slide the original bracket along the cable and tuck the end into the Velcro strap at the back of the frame.



7. Ensure that the cable clip is located on the frame, as shown above.
8. Reconnect the battery.

Caution! We recommend that the speed setting is set to Low, whilst you become familiar with using the attendant controller.

Lefthand Mounting:

The attendant controller bracket can also be mounted on the lefthand push handle. In this case, the controller will be positioned on the outside of the push handle, as opposed to being on the inside, when mounted on the righthand push handle.

Armrest Mounting:

To move the controller back to the armrest:

- a) Detach the attendant controller bracket from the push handle.
- b) Remove the attendant controller bracket from the controller.
- c) Refit the original bracket to the controller.
- d) Secure the bracket into the armrest, by tightening the knob.
- e) Ensure that the cable is safely routed and clipped onto the frame.